

## **BRITEVENT®**

## **Product Data Sheet**

**Product Description: Britevent Urine Temperature Indicators** 

Standard Range: 88 to 102°F or 32 to 39°C

## Construction:

Adhesive Tape	Double-coated Adhesive Tape
Covering Film	Polyester 3 mil stock
Color Change Material	Non toxic mixture of liquid crystal esters
Accuracy	±1°F / ± 0.6°C
Shelf Life	12 months from invoice date when stored at room temperature (i.e. 21°C (70°F) & 50% relative humidity).

Application Data: Apply temperature indicating label to outside of urine specimen cup. Read

temperature after colors stop changing. Green is correct reading. If green is not

visible, subtract 1°F from highest lit event.

**Function:** Display of temperature, as indicated by green event, of urine specimen cup. Label

is available in direct read or 2°F offset forms.

**Use:** Ideal for use in clinics, hospitals, and doctors' offices.